PL

# ALUSTAT

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EN

# ALUSTAT

Liquid for staunching the bleeding

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY**

**COMPOSITION:** aluminum chloride 25%, water Ph.Eur., menthol, excipients.

#### INTENDED USE

Product ALUSTAT is intended for use during dental treatment to staunch slight gingival bleedings. Aluminum chloride contained in the product acts vasoconstrictively on tiny blood vessels, as a result the gingival swelling and slight bleedings decrease.

#### INSTRUCTIONS FOR USE

Swollen or injured gingiva rub gently with cotton wool tampon impregnated with ALUSTAT, next rinse it with clean water. Repeat action if necessary.

#### CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

#### PRECAUTIONS

It is irritating to the mucous membrane of the oral cavity, skin and eyes. In case of contact rinse immediately with copious amount of water and seek medical advice. In case of consumption do not extract emesis. Drink copious amount of water. Seek medical advice. Necessary use of disposal gloves and protective glasses while working with the product.

#### STORAGE

Store in original packaging at temperature below 25 °C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

#### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

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