PL

ALUSTAT gel

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ALUSTAT gel

Gel for staunching bleeding

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: aluminum chloride 25%, water Ph.Eur., excipients.

INTENDED USE

Product ALUSTAT GEL is intended for use during dental treatment to staunch slight gingival bleedings. Aluminum chloride contained in the product acts vasoconstrictively on tiny blood vessels, as a result the gingival swelling and slight bleedings decrease.

INSTRUCTIONS FOR USE

Put the supplied applicator in ethanol before use, then install the applicator on the product syringe. Apply the product directly on the bleeding spot and leave for about 30 seconds, next rinse it with clean water. Repeat if necessary. Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS

It is irritating to the mucous membrane of the oral cavity, skin and eyes. In case of contact rinse immediately with copious amount of water and seek medical advice. In case of consumption do not induce vomiting. Drink copious amount of water. Seek medical advice. Use disposal gloves and protective glasses while working with the product.

STORAGE

Store in original packaging at temperature below 25 °C. Store out of reach of children. Expiry date is placed on the main container. Once the container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Syringe of 5 ml or 10 ml, dispenser, connector, disposal applicators.

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