PL

BLUE ETCH *Flow*

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BLUE ETCH *Flow*

Flowable dental etchant

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY

COMPOSITION: o-phosphoric acid 36%, gel base, dyestuff.

INTENDED USE

BLUE ETCH FLOW product is used for etching dentine and enamel prior to composites, fissures and cavity seals. Thanks to its carefully formulated semi-solid consistence, BLUE ETCH FLOW reaches inaccessible dental grooves and fissures. The product is also recommended for use before cementing glass fibre crown and root inlays.

PREPARATION OF THE PRODUCT BEFORE USE

(concerning: variant 10 ml and 50 ml)

1. Unscrew the white plug from the dosing syringe containing etchant.
2. Replace white plug with attached connector.
3. Unscrew the blue plug from the connector and replace it with dispenser – smaller syringe attached to the product.
4. Push down the plunger of dosing syringe and pump required amount of gel into the dispenser, but no more than 2ml.
5. Unscrew the filled dispenser and screw attached applicator onto the top (applicator needle with blunt end). The product is ready for use.
6. Leave the connector screwed on the dosing container and protect it with the blue plug.

INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator into the spirit (ethyl alcohol) and then place it onto the syringe. Apply a thin layer of the etchant onto the dried surface. Etch no longer than 60 seconds. Rinse with strong air-flow and water (for about 15-20 seconds). Do not dry too much. In case of enamel impurities repeat the process. Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS

It is irritating to the mucous membrane, skin and eyes. In case of contact rinse immediately with copious amount of water and seek medical advice. In case of consumption do not extract emesis. Drink copious amount of water. Seek medical advice. Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and protective clothing.

STORAGE

Store in original packaging at temperature below 25°C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Syringe of 2ml of preparation, disposable applicator. Syringe of 10 ml or 50 ml with preparation, disposable applicators, connector, dispenser.

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