PL

## CHLORAXID 5,25% EXTRA

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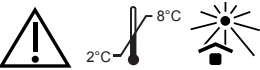
## CHLORAXID 5,25% EXTRA

Liquid for root canals rinsing

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** sodium hypochlorite (5,25% of active chlorine), water Ph. Eur., surfactants.

### INTENDED USE

During mechanical canal widening it removes the non-vital pulp debris. It cleans the canal, removes the smear layer in order to expose dentinal tubule orifices before canal filling. It prevents from the teeth discoloration that might occur after filling.

### Adapter with a plug for filling the syringe

- Unscrew the bottle cap.
- Screw adapter on the bottle.
- Remove the blue plug, screw the end of the syringe into the adapter's opening. Turn the bottle upside down and take the liquid into the syringe, then turn the bottle upright on the table top and gently unscrew the filled syringe.
- It is recommended to use luer-lock syringes with rubber plunger.
- After each use close the adapter's opening with a blue plug.
- For transportation remove adapter and replace the bottle cap.

### INSTRUCTION FOR USE

Insert the liquid into the root canal by means of the syringe with a needle with side port. Rinse the canal copiously (approximately 10ml for each canal). Sodium hypochlorite contained in the product inactivates immediately in contact with organic substance. While concentration of sodium hypochlorite is decreasing, time of rinsing and volume should be necessarily simultaneously increased.

### CONTRAINDICATIONS

Do not use product on patients with hypersensitivity to the product's components.

**CAUTION!** While rinsing root canals, it is compulsory to rinse with physiological saline between rinsing sodium hypochlorite and chlorhexidine digluconate.

It prevents from precipitating of a tawny deposit. Rinse copiously.

### PRECAUTIONS

During root canal preparation do not use too heavy stream of the liquid because it can cause the liquid penetration to the periapical tissues and damage them, characterized by pain and swelling of soft tissues. Special care should be taken when product applied into perforations. Irritating to the mucous membranes of the mouth, eyes and skin. In case of contact, rinse immediately with plenty of water and seek for medical advice.

In case of consumption do not extract emesis. Drink plenty of water. Seek for medical advice. Use rubber-dam while working with the product. Recommended use of personal protective equipment (glasses, protective visor, gloves).

**DISPENSER IS NOT INTENDED FOR THE LIQUID STORAGE!**

### STORAGE

Store in the original containers at temperature 2 – 8 °C (in refrigerator). Protect the product from the sunlight. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

### TRANSPORTATION

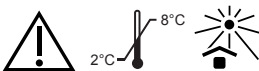
The product manufacturer recommends that the transportation period is no longer than 10 days at temperature not exceeding 40 °C. Short-term transportation within the specified temperature range do not affect the safety and efficacy of the product throughout the shelf life of the product for use.

### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to manufacturer.

### PACKAGE

Bottle of 200 g or 400 g of the preparation, adapter with screw cap, dispenser.

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