PL

Eugenol

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EN

Eugenol

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: eugenol oil 100%

INTENDED USE

Product EUGENOL is used as a liquid for making pastes for root canal filling and temporary filling of cavities. Mainly used with zinc oxide.

INSTRUCTIONS FOR USE

For temporary filling of cavities consistency of the mixture must be made so as it doesn't stick neither to the glass container nor to the spatula. For root canal filling received mixture must have creamy consistency.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components. Do not use under composites since they may disturb polymerization process. Do not use the product for pregnant women.

PRECAUTIONS

May cause an allergic skin reaction. It may cause irritation to the mucous membrane of the oral cavity and eyes. In case of contact rinse immediately with water and seek medical advice. In case of consumption drink plenty of water, don't extract emesis. Seek medical advice. Use rubber-dam while working with the product. Use gloves, protective glasses and clothing.

STORAGE

Store in original packaging at temperature below 25 °C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Used packaging should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Bottle with dropper of 10 ml or 20 ml of preparation.

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