PL

GLUGO-CHeX 2%

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GLUGO-CHeX 2%

Liquid for root canals rinsing

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: chlorhexidine digluconate 2%, water Ph. Eur.

INTENDED USE

Liquid is used for rinsing of root canals in order to remove debris. It improves mechanical preparation of root canals. It lubricates a root canal during mechanical preparation. **Adapter with a plug for taking the liquid preparations into the syringe**

1. Unscrew the bottle cap.
2. Screw the adapter on the bottle.
3. In order to use the bottle remove the blue plug, screw the end of the syringe into the adapter's opening, slope the bottle with preparation upside down and take the liquid into the syringe, while holding down the adapter at the same time. Next put the bottle upright on the table top and gently unscrew the filled syringe from adapter. It is recommended to use luer-lock syringes with a rubber plunger.
4. After each use close the adapter's opening with a blue plug.
5. For transportation remove the adapter and twist the bottle with the bottle cap.

INSTRUCTION FOR USE

Insert the liquid into the root canal using a syringe with a side vent needle during mechanical preparation. Rinse the canals copiously, approximately 10 ml for one canal.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

CAUTION! While rinsing root canals, it is necessary to rinse with physiological saline between rinsing sodium hypochlorite and chlorhexidine digluconate. It prevents from precipitating of a tawny deposit. Rinse copiously.

PRECAUTIONS

Product may cause irritation of the mucous membrane of the oral cavity and eyes. In case of contact rinse with water immediately. In case of consumption drink copious amount of water. Do not induce vomiting. Seek medical advice. Use rubber-dam while working with the product. Recommended use of personal protective equipment (glasses, protective visor, gloves). **DISPENSER IS NOT INTENDED FOR THE LIQUID STORAGE!**

STORAGE

Keep out of reach of children. Protect from direct sunlight. Expiry date is placed on the main container. Store in the original container at temperature below 25 °C. Once the container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Bottle of 200 g or 400 g of the preparation, adapter with screw cap, dispenser.

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