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Material for filling and rebuilding of root canals

PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE FOR DENTAL USE ONLY

INGREDIENTS:

MTA+ powder: calcium oxide, oxides of: silicon, iron, aluminium, sodium, potassium, bismuth, magnesium, zirconium; calcium phosphate.

INDICATIONS FOR USE

The product is intended for dental treatment as a material for filling and remineralization of root canals. The compound cures in 6 to 10 minutes after mixing with water.

The high content of calcium ions remineralizes the tooth tissue. The silicon and calcium compounds support tissue regeneration after wall perforation and intracanal resorption.

MTA+ is especially recommended in:

- root canal wall perforation,

- intracanal resorption,

- filling of the posterior root apex,

- direct coating of the pulp,

- pulp amputation,

- treatment of teeth with interrupted root development.

MTA+ is not suitable as a standalone permanent root canal filling.

INSTRUCTIONS FOR USE

Cut off the *MTA+ liquid* pipette tip. Move the entire contents of the *MTA+ powder* glass vial with 1 to 2 drops of the *MTA+ liquid* on a mixing plate. Mix for ca. 30 s until the compound reaches a consistency of moist sand. If the compound mix is too thick or too brittle, add one more drop of the liquid (or deionized water). Place the compound at the target site with an applicator.

Exposure to humidity, also from blood, will reduce the curing time.

One container of the *MTA+ powder* contains a one-off dose.

Once mixed with the *MTA+ liquid* (or deionised water) consume the mixed dose within a maximum of 10 minutes. The compound cures and hardens afterwards. Consume the contents of the *MTA+ liquid* container in 12 months after opening.

CONTRAINDICATIONS

Do not use in patients who are hypersensitive to any of the ingredients. Do not use in inflammatory conditions. Acidic pH values disrupt setting of *MTA+*.

PRECAUTIONS

The product can cause eye damage.

The product irritates the oral mucosa, skin and eyes. In case of contact, rinse with plenty of water and contact a doctor immediately.

If swallowed, do not induce vomiting. Drink plenty of water. Contact a doctor immediately. Use rubber dams, disposable gloves and protective visors when working with the product.

STORAGE

Store in a tightly closed original container below 25 °C. Protect from moisture.

Keep out of the reach of children.

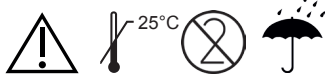
The expiry date is on the direct packaging.

DISPOSAL OF PACKAGING

Waste containers should be disposed of or returned to the manufacturer.

PACKAGING

Glass vial containing 0.14 g of the *MTA+ powder*; pipette containing 1 ml of the *MTA+ liquid*.

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© 2016 MTA+ s.p.a. - All rights reserved. - MTA+ is a registered trademark.