PL

RED DETECTOR

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RED DETECTOR

Caries detector

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: 4-ethylo rhodamine, propylene glycol, surfactants.

INTENDED USE

Product RED DETECTOR is intended for locating the caries lesion. Red dye contained in the product invades into the tissue with caries lesion and dyes it red. RED DEECTOR increases safety of work by locating the carious dentine and saving the healthy tissues.

INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator into the spirit (ethyl alcohol) and then place it onto the syringe. After removing all visible carious lesions apply one drop of preparation on the defect area, wait 5-10 seconds, next irrigate and dry the cavity. Points intensive dyed in red indicate carious dentine. After product use withdraw the syringe piston in order to prevent overflow of preparation, take off applicator and screw the white plug into the syringe. Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS

Use rubber-dam while working with the product. Use disposal gloves, protective glasses and clothing.

STORAGE

Store in original packaging at temperature below 25° C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or return to the manufacturer.

PACKAGING

Syringe of 2 ml of preparation, disposal brush applicators.

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