PL

ZINC OXIDE

Tlenek cynku

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ZINC OXIDE

Zinc oxide

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: 100% zinc oxide of pharmacopeial purity.

INTENDED USE

Product ZINC OXIDE is recommended for making pastes, mainly with Eugenol, used as material for temporary filling of cavities and root canals. Mixture of zinc oxide with eugenol as temporary filling has drying effect and stimulates creation of the secondary dentine.

INSTRUCTIONS FOR USE

- For temporary filling of the defects mixture must be made in such a way that it cannot stick to the glass container nor to the spatula.
- For temporary filling of the root canals prepared mixture should have creamy consistency. Leave the filling in cavity or root canal for a period of time from 1 week up to 30 days.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS

It may cause eyes irritation. In case of contact rinse immediately with copious amount of water and seek medical advice. In case of consumption drink copious amount of water, do not extract emesis. Seek medical advice. Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and protective clothing.

STORAGE

Store in original packaging at temperature below 25 °C. Protect from moisture. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

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