PL

# RAINBOW flow

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# RAINBOW flow

Flowable composite

**PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE PRODUCT FOR DENTAL USE ONLY**

**INGREDIENTS:** methacrylic resins (bis-GMA, UDMA ,TGDMA), inorganic nanoparticle filling, pigments, photoinitiator, inhibitor.

#### INDICATIONS FOR USE

The RAINBOW FLOW light-curing flow composite product is intended for:  
- filling of milk teeth and permanent teeth  
- pit and fissure sealing  
- marking of root canal orifices

#### INSTRUCTIONS FOR USE

Put the supplied applicator in ethanol before use, then install the applicator on the product syringe.

#### Filling and pit and fissure sealing:

Prepare and etch the defect/pit/fissure (e.g. with BLUE ETCH), then rinse and dry. Use a bonding system. Fill the defect with the composite layer ca. 2 mm thick, then cure with a polymerization lamp for ca. 10 to 15 seconds. Add more of the composite if needed and cure.

#### Marking of root canal orifices:

Apply after permanently filling the root canal. Apply the composite layer ca. 2mm thick to the root canal orifice, then cure with a polymerization lamp for ca. 10 to 15 seconds. The bright colour of the composite helps precisely locate the orifices of sealed root canals during follow-ups.

Retract the composite syringe piston and seal the container with the original cap immediately after use to avoid polymerization of the product inside.

The product is intended for repeated use until spent. The supplied applicator is disposable. Reusing the applicator may cause a risk of infection.

#### CONTRAINDICATIONS

Do not use in patients who are hypersensitive to any of the ingredients.

#### PRECAUTIONS

This product can cause eye irritation in the uncured form. In case of contact, rinse with plenty of water and contact a doctor immediately. If swallowed, drink plenty of water. Do not induce vomiting. Contact a doctor immediately.  
Use rubber dams, disposable gloves, safety goggles and protective clothing when working with this product.

#### STORAGE

Store in a tightly closed original container below 25 °C.

Protect from direct sunlight. Reseal after use.

Keep out of the reach of children.

The expiry date is on the direct packaging.

Opening the originally sealed container does not change the expiry date if the product is resealed after each use.

#### DISPOSAL OF PACKAGING

Waste containers should be disposed of or returned to the manufacturer.

#### PACKAGING

Syringe, containing 1g of the preparation and a pack of disposable applicators.

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