PL

**CHLORAXID 5,25%
GEL EXTRA**

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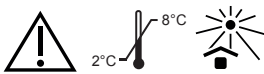
EN

**CHLORAXID 5,25%
GEL EXTRA**

Preparation for root canals rinsing

PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE PRODUCT FOR DENTAL USE ONLY

INGREDIENTS: sodium hypochlorite (5.25% of active chlorine), Ph Eur purified water, surfactants, gel base.

INDICATIONS FOR USE

Removal of dead pulp residues during mechanical broaching of root canals. Cleans the root canal, including the synovial layer to expose the dental tubule orifices prior to filling the canal. Prevents discolouration of the teeth from filling a root canal without flushing. The gel consistence of the preparation minimises the risk of feeding the product beyond the apical opening and ensures a precise control of application. CHLORAXID 5.25% GEL EXTRA is specifically recommended for preparing root canals with wide, underdeveloped or damaged apexes.

INSTRUCTIONS FOR USE

Put the supplied applicator in ethanol before use, then install the applicator on the product syringe. Prepare the root canal by feeding CHLORAXID 5.25% GEL EXTRA directly into the canal with the supplied applicator, or coat the intracanal instrument of choice with the product. Prepare the root canal and flush out with normal saline fed via a needle with a lateral orifice. Flush alternately with EDTA. Repeat several times. The product is intended for repeated use until spent. The supplied applicator is DISPOSABLE. Reusing the applicator may cause a risk of infection.

CONTRAINDICATIONS

Do not use in patients who are hypersensitive to any of the ingredients. **CAUTION!** Flushing root canals: separate the sodium hypochlorite flush and the chlorhexidine digluconate flush with a normal saline flush to prevent a brownish sediment from forming. Thoroughly flush and rinse.

PRECAUTIONS

Extreme caution is advised when perforation is present. The product may irritate the oral mucosa, the skin and eyes. In case of contact, rinse with plenty of water and contact a doctor immediately. If swallowed, do not induce vomiting. Drink plenty of water. Contact a doctor immediately. Use rubber dams when working with the product. Use disposable gloves, safety goggles and protective clothing.

STORAGE

Store in the original container at 2 to 8 °C (refrigerate). Protect from direct sunlight. Keep out of the reach of children. The expiry date is on the direct packaging. Opening the originally sealed container does not change the expiry date if the product is resealed after each use.

TRANSPORT

The product can be shipped for a maximum of 10 days under 40°C. Short distance shipping and handling within the recommended temperature range does not affect the safety of use and efficacy of the product before its expiry date.

DISPOSAL OF PACKAGING

Waste containers should be disposed of or returned to the manufacturer.

PACKAGING

Syringe, containing 2 ml of the preparation and a pack of disposable applicators.

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