PL

## CHLORAXID 2%

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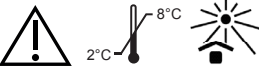
## CHLORAXID 2%

Liquid for root canals rinsing

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** sodium hypochlorite (2% of active chlorine), water Ph. Eur.

### INTENDED USE

During mechanical canal widening it removes the non-vital pulp debris. It cleans the canal, removes the smear layer in order to expose the dentin root canal orifices before final filling.

It prevents from the tooth discoloration that might occur after filling the non-rinsed root canal.

**Adapter with a plug for taking the liquid preparations into the syringe**

1. Unscrew the bottle cap
2. Screw adapter on the bottle.
3. In order to use the bottle remove the blue plug, screw the end of the syringe into the adapter's opening, slope the bottle with preparation upside down and take the liquid into the syringe, while holding down the adapter at the same time. Next put the bottle upright on the table top and gently unscrew the filled syringe from adapter. It is recommended to use luer-lock syringes with rubber plunger.
4. After each use close the adapter's opening with a blue plug.
5. For transportation remove adapter and twist the bottle with a bottle cap.

### INSTRUCTION FOR USE

Insert the liquid CHLORAXID 2% into the root canal by means of the syringe with a needle with side portal. Rinse the canal copiously, approximately 10 ml for one canal. Sodium hypochlorite contained in the product inactivates immediately in contact with organic substance. While concentration of sodium hypochlorite is decreasing, time of rinsing and volume should be necessarily simultaneously increased.

### CONTRAINDICATIONS

Do not use product CHLORAXID 2% for patients with hypersensitivity to the product's components.

**CAUTION!** While rinsing root canals, it is compulsory to rinse with physiological saline between rinsing sodium hypochlorite and chlorhexidine digluconate.

It prevents from precipitating of a tawny deposit. Rinse copiously.

### PRECAUTIONS

During root canal preparation do not use too heavy stream of the liquid because it can cause the liquid penetration to the periapical tissues and damage them, characterized by pain and swelling of soft tissues. Special care should be taken when product applied into perforations. Irritating to the mucous membranes of the mouth, eyes and skin. In case of contact, rinse immediately with plenty of water and seek for medical advice.

In case of consumption do not extract emesis. Drink plenty of water. Seek for medical advice. Use rubber-dam while working with the product. Recommended use of personal protective equipment (glasses, protective visor, gloves).

**DISPENSER IS NOT INTENDED FOR THE LIQUID STORAGE!**

### STORAGE

Store in the original containers at temperature 2 – 8 °C (in the refrigerator). Store in a securely closed packaging. Protect the product from the sunlight. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

### TRANSPORTATION

The product manufacturer is admitting the product transportation within the period not longer than 10 days at temperature not exceeding 40 °C. Short transportation period at given temperature range has no influence on safety and effectiveness of the product throughout its shelf life.

### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

### PACKAGING

Bottle of 200 g or 400 g of preparation, adapter with screw cap, dispenser.

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### MODE D'EMPLOI

Insert the liquid CHLORAXID 2% into the root canal by means of the syringe with a needle with side portal.

Rinse the canal copiously, approximately 10 ml for one canal.

Sodium hypochlorite contained in the product inactivates immediately in contact with organic substance.

While concentration of sodium hypochlorite is decreasing, time of rinsing and volume should be necessarily simultaneously increased.

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