PL

CHLORAXID 5,25%

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CHLORAXID 5,25%

Liquid for root canals rinsing

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY

PRODUCT FOR DENTAL USE ONLY

COMPOSITION:

sodium hypochlorite (5.25% of active chlorine), water Ph. Eur.

INTENDED USE

During the mechanical canal widening it removes the remains of dead pulp. It cleans the canal and removes the smear layer to expose the dentin root canal orifices before final filling. It prevents discoloration of the teeth that might occur after filling the non-rinsed root canal. Adapter with a plug for taking the liquid preparations into the syringe 1. Unscrew the bottle cap. 2. Screw the adapter on the bottle. 3. In order to use the bottle, remove the blue plug, screw the end of the syringe into the adapter opening, tilt the bottle with the preparation upside down and take the liquid into the syringe, while simultaneously holding down the adapter. Next, put the bottle upright on the table top and gently unscrew the filled syringe from adapter. It is recommended to use luer-lock syringes with a rubber plunger. 4. After each use, close the adapter opening with a blue plug. 5. For transport, remove the adapter and screw the cap on the bottle.

CONTRAINDICATIONS

Do not use the product CHLORAXID 5.25% for patients with hypersensitivity to the product's components. **CAUTION!** While rinsing root canals, use physiological saline between rinsing with sodium hypochlorite and chlorhexidine digluconate. It prevents the precipitation of a brown deposit. Rinse copiously.

PRECAUTIONS

During root canal preparation do not use a too heavy stream of the liquid because it can cause the liquid to penetrate the periapical tissues and damage them, resulting in pain and swelling of soft tissues. Special care should be taken when the product is applied into perforations. Irritating to the mucous membranes of the mouth, eyes and skin. In case of contact rinse immediately with plenty of water and seek medical advice. In case of consumption do not induce vomiting. Drink plenty of water. Seek medical advice. Use rubber-dam while working with the product. The use of personal protective equipment (glasses, protective visor, gloves) is recommended. **DISPENSER IS NOT INTENDED FOR LIQUID STORAGE!**

STORAGE

Store in original packaging at 2-8 °C (in a refrigerator). Store in a securely closed packaging. Protect from sunlight. Keep out of reach of children. Expiry date is placed on the main container. Once the container is open, the expiry date doesn't change if it the container is securely closed after each use.

TRANSPORT

The manufacturer allows transport of the product for a period not longer than 10 days at a temperature not exceeding 40 °C. Short-term transport within the given temperature range does not affect the safety and efficacy of the product throughout its shelf life.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer

PACKAGING

Bottle of 200 g or 400 g of preparation, adapter with a screw cap, dispenser.

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