PL

ENDO-PREP gel

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ENDO-PREP gel

Gel for root canals preparation

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
 PRODUCT FOR DENTAL USE ONLY

COMPOSITION: disodium edetate (EDTA 17%), gel base.

INTENDED USE

Product ENDO-PREP GEL is intended for chemo-mechanical root canal preparation. EDTA contained in the product helps to remove calcium salts and pulp from decalcified root canals, enabling for faster cleansing and shaping the canals. Thanks to the lubricant substances contained in the product, it is much easier to work with the root canal instruments and the risk of breakage of the instrument in the root canal is minimized.

INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator into ethyl alcohol and then place it onto the syringe. Open the tooth cavity and rinse it with sodium hypochlorite. Insert the preparation into tooth cavity by means of attached applicator or place it directly on the root canal instrument. Prepare the canal mechanically while rinsing it with sodium hypochlorite simultaneously. Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use ENDO PREP GEL for patients with hypersensitivity to the product's components. Do not leave ENDO PREP GEL in the canal or in the tooth cavity between the visits. It may soften the dentin too much.

PRECAUTIONS

Caution should be taken during canal preparation close to the apex area, because it may cause perforation of the canal as a result of use of root canal instruments. Eye and skin irritant. In case of contact, rinse immediately with plenty of water and seek for medical advice. In case of consumption drink copious amount of water. Do not extract emesis. Seek for medical advice. Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and clothing.

STORAGE

Store in original packaging at temperature below 25 °C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Syringe of 2 ml, 5 ml or 10 ml of preparation, disposable applicators.

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