**CHLORAXID 5,25%  
EXTRA**

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**PRECAUTIONS** During root canal preparation do not use a too heavy stream of the liquid because it can cause the liquid to penetrate the periapical tissues and damage them, resulting in pain and swelling of soft tissues. Extreme caution is advised when working with perforations.

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Once container is open, the expiry date does not change if the container is securely closed after each use.

**TRANSPORT:** The product can be shipped for a maximum of 10 days under 40 °C. Short distance shipping and handling within the recommended temperature range does not affect the safety of use and efficacy of the product before its expiry date.

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**CHLORAXID 5,25%  
EXTRA**

Liquid for root canals rinsing

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** sodium hypochlorite (5,25% of active chlorine), water Ph. Eur., surfactants.

**INTENDED USE:** Removal of dead pulp residues during mechanical preparation of the root canal. Cleans the root canal, including the synovial layer to expose the dental tubule orifices prior to filling the canal.

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**PREVENTION:** Prevents discoloration of the teeth that might occur after filling the non-rinsed root canal.

**ADAPTER WITH A PLUG FOR TAKING THE LIQUID PREPARATIONS INTO THE SYRINGE:**

1. Unscrew the bottle cap.
2. Screw the adapter on the bottle.
3. Remove the blue plug, screw the end of the syringe into the adapter's opening. Turn the bottle upside down and pull the liquid into the syringe, while simultaneously holding the adapter. Next, put the bottle upright on the table top and gently unscrew the syringe from the adapter. Use Luer-lock syringes with a rubber stopper.
4. After each use, close the adapter's opening with a blue plug.
5. For transport, remove the adapter and close the bottle with a bottle cap.

**INSTRUCTIONS FOR USE:**

Insert the liquid into the root canal using a needle with an endodontic opening. Rinse thoroughly (approximately 10ml for each canal). Sodium hypochlorite in the product inactivates immediately in contact with organic substance. As the concentration of sodium hypochlorite decreases, the rinsing time and the volume of the solution used should be increased.

**CONTRAINDICATIONS:** Do not use for patients who are hypersensitive to the product components.

**CAUTION!** Flushing root canals: separate the sodium hypochlorite flush and the chlorhexidine digluconate flush with a normal saline flush to prevent a brownish sediment from forming. Thoroughly flush and rinse.

**PRECAUTIONS:** During root canal preparation do not use a too heavy stream of the liquid because it can cause the liquid to penetrate the periapical tissues and damage them, resulting in pain and swelling of soft tissues. Extreme caution is advised when working with perforations.

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