## PL RUBBER-DAM liquid

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## EN RUBBER-DAM liquid

Light-cured rubber-dam liquid

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** urethane resins, inorganic fillers

**INTENDED USE**

Light-cured rubber-dam liquid is intended for:  
- gingival protection during dental treatments: whitening, etching, microabrasion, sandblasting etc.  
- sealing rubber dams  
Suitable for occlusal stamp technique.

**INSTRUCTIONS FOR USE**

Before the treatment immerse attached applicator in alcohol (ethyl alcohol) and then fix it onto the syringe.

For gingival protection during dental treatments:  
Clean and dry the gingiva. With the use of appropriate instruments (e.g. buccal and labial retractor) immobilize the lips and tongue in order to avoid contact with the liquid. Remove the syringe lock and install the included disposable applicator. Apply RUBBER-DAM LIQUID along the application area 4-6 mm wide and approximately 1.5-2 mm thick, at the same time visually checking the amount of the applied preparation. If possible, squeeze RUBBER-DAM LIQUID through the interdental space. Cure the preparation for 20-30 seconds using a polymerisation lamp. Carry out the curing process evenly in a scanning motion. Proceed to the dental treatment. After the treatment is completed using an instrument from one side, lift off and remove polymerised RUBBER-DAM LIQUID.

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In order to seal the rubber dam, apply a layer of the liquid on the edge of the gingiva and the rubber dam. Cure the preparation for 20-30 seconds using a polymerisation lamp. Carry out the curing process evenly in a scanning motion. Proceed to the dental treatment. After the treatment is completed, using an instrument from one side, lift off and remove polymerised RUBBER-DAM LIQUID.

Immediately after use, withdraw the syringe plunger and place the original cap on the container to avoid polymerisation of the product in the syringe. Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

**CONTRAINDICATIONS**  
Do not use the product for patients with hypersensitivity to the product's components.

**PRECAUTIONS**  
Un-cured substance can cause irritation to the skin and eyes. Sustained contact with the skin can cause allergic reaction. In case of contact, immediately rinse with plenty of water. Harmful when swallowed. If swallowed, do not induce vomiting. Contact a doctor immediately.

**STORAGE**  
Store in a tightly closed original container below 25 °C. Protect against direct sunlight. Close tight after use. Keep out of reach of children. The expiry date is on the direct packaging. Once container is open, the expiry date doesn't change if the container is securely closed after each use.

**HANDLING THE PRODUCT CONTAINER**  
Empty containers should be disposed of responsibly or returned to the manufacturer.

**PACKAGING**  
Syringe containing 1.2 ml of the preparation, disposable applicators.

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