PL

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EN

CALCIPAST+I

Calcium hydroxide paste with iodoform

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: calcium hydroxide, iodoform, barium sulphate, excipients.

INTENDED USE
CALCIPAST+I is intended for use in dental treatment for temporary root canal filling.
It supports restoration of damaged periapical tissues. Iodoform in the product has a drying effect on root canal exudates. pH≈11.
Radiopaque.

INSTRUCTIONS FOR USE
Immerse the supplied applicator in ethanol before use, then install the applicator on the product syringe.
Measure the canal length, adjust the endstop on the applicator to the length 2-3 mm shorter than the measured canal length. Apply the paste into the canal, remove excess paste and close the cavity securely. Leave the filling in the root canal for 1 week up to 30 days. A reusable product with a single-use applicator.
Re-using the applicator may cause the risk of reinfection.

CONTRAINDICATIONS
Do not use for patients who are hypersensitive to the product components.

PRECAUTIONS
Harmful if swallowed. Irritating to the mucous membranes of the oral cavity and skin. May cause serious eye damage.
In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting.
Drink plenty of water.
Seek medical advice. Use a rubber-dam when working with the product.
Use protective gloves, glasses and protective clothing.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Keep in original packaging below 25 °C.
Keep out of reach of children.
Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.
Protect from sunlight.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Syringe containing 2.1 g of the preparation, disposable applicators.

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