PL	Preparat do płukania kanałów korzeniowych	RU	Препарат для полоскания корневых каналов
EN	Preparation for root canals rinsing	IT	Preparato per lavaggi canali
ES	Producto para enjuagar conductos radiculares	RO	Produs destinat spălării canalelor radiculare
FR	Préparation pour le rinçage des canaux radiculaires	HU	Termék a gyökércsatorna öblítéséhez
DE	Präparat zur Spülung der Wurzelkanäle	BG	Препарат за изплакване на каналите на корените
		LT	Šakny kanalų skalavimo preparatas
Data ostatniej aktualizacji: 18.07.2016/1			

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EN

Preparation for root canals rinsing

PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE
PRODUCT FOR DENTAL USE ONLY

INGREDIENTS: sodium hypochlorite (5.25% of active chlorine), Ph Eur purified water, gel base.

INDICATIONS FOR USE
Removal of dead pulp residues during mechanical broaching of root canals. Cleans the root canal, including the synovial layer to expose the dental tubule orifices prior to filling the canal. Prevents discolouration of the teeth from filling a root canal without flushing. The gel consistence of the preparation minimises the risk of feeding the product beyond the apical opening and ensures a precise control of application. CHLORAXID 5.25% GEL is specifically recommended for preparing root canals with wide, underdeveloped or damaged apexes.

INSTRUCTIONS FOR USE
Put the supplied applicator in ethanol before use, then install the applicator on the product syringe. Prepare the root canal by feeding CHLORAXID 5.25% GEL directly into the canal with the supplied applicator, or coat the intracanal instrument of choice with the product. Prepare the root canal and flush out with normal saline fed via a needle with a lateral orifice. Flush alternately with EDTA. Repeat several times. The product is intended for repeated use until spent. The supplied applicator is DISPOSABLE. Reusing the applicator may cause a risk of infection.

CONTRAINDICATIONS
Do not use in patients who are hypersensitive to any of the ingredients. **CAUTION!** Flushing root canals: separate the sodium hypochlorite flush and the chlorhexidine digluconate flush with a normal saline flush to prevent a brownish sediment from forming. Thoroughly flush and rinse.

PRECAUTIONS
Extreme caution is advised when perforation is present. The product may irritate the oral mucosa, the skin and eyes. In case of contact, rinse with plenty of water and contact a doctor immediately. If swallowed, do not induce vomiting. Drink plenty of water. Contact a doctor immediately. Use rubber dams when working with the product. Use disposable gloves, safety goggles and protective clothing.

STORAGE
Store in the original container at 2 to 8 °C (refrigerate). Protect from direct sunlight. Keep out of the reach of children. The expiry date is on the direct packaging. Opening the originally sealed container does not change the expiry date if the product is resealed after each use.

TRANSPORT
The product can be shipped for a maximum of 10 days under 40°C. Short distance shipping and handling within the recommended temperature range does not affect the safety of use and efficacy of the product before its expiry date.

DISPOSAL OF PACKAGING
Waste containers should be disposed of or returned to the manufacturer.

PACKAGING
Syringe, containing 2 ml of the preparation and a pack of disposable applicators.

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