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INTENDED USE

Protective shields for a patient prevent injuries of an eye and eye area that could happen as a result of fairly harmless mechanical conditions that occur during dental procedures.

METHOD OF ASSEMBLY

Remove the foil gently, avoid causing scratches and contact with items likely to cause damage. Connect the endings of foils with the clips from both sides of the visor.

Fix the foil when frame is unfolded.

The foil can be individually adjusted to suit the user.

Do not fold the spectacle frame with the foil attached; take off the foil before folding.

NOTICE: Visors are made of thermoplastic material. To shape the visor to a desired shape, soak in hot water beforehand and then shape as required.

MAINTENANCE

To maintain the visor, remove foil from the spectacle frame and rinse it carefully with water and detergent, then leave it to dry. Foil can be used once again. If any scratches or blurring occur as result of use, discard the foil and use another sheet of replaceable foil.

Don't wipe!

Don't use spirit or an aggressive cleaning agent!

Manufacturer does not take any responsibility for using the product in a manner not in accordance with instructions or intended use.

CONTENTS OF SET

Spectacle frame with plastic clips + set of replaceable foils.

PPH CERKAMED Wojciech Pawłowski

37-450 Stalowa Wola, Polska

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Tel/fax 0048 15 842-35-85