PL

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EN

Porcelain etch

ES

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YELLOW PORCELAIN ETCH

Porcelain etch

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: hydrofluoric acid 9,5%, base gel, dye.

INTENDED USE
Product is used for porcelain etching during reparations of the dentures and as well as for etching of the porcelain veneers, crowns or inlays before their cementation.

INSTRUCTIONS FOR USE
Before the treatment immerse attached applicator into the spirit (ethyl alcohol) and then place it onto the syringe.
1.Isolate operation field with rubber-dam.
2. Screw elastic blue applicator on the syringe with the etchant, replace safety plug with this applicator.
3. Apply small amount of etching preparation on the prepared and dried surface. After use screw the syringe with a safety plug.
4. Leave etching for 60 seconds and then rinse and dry the procedure spot.
5. Apply Silan according to its instruction for use provided by producer.
Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

CONTRAINDICATIONS
Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS
Corrosive preparations – avoid contact with skin and mucosa membrane, in case of contact rinse immediately with copious amount of running water, seek medical advice.
Avoid contact with eyes, in case of contact rinse with plenty of water with wide open lid, seek ophthalmologist advice.
In case of consumption do not extract the emesis, seek medical advice.
In case of any reparations of the dentures directly in the patient's oral cavity it is required to use rubber-dam and any protection for patient's eyes. Use vacuum system while etching. Equip personnel with protective clothing, gloves and visors.

STORAGE
Store in original packaging at temperature below 25 °C. Store out of reach of children.
Expiry date is placed on the main container.
Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Syringe of 2 ml of preparation, elastic applicators.

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