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BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY
NON - STERILE PRODUCT STERILIZE DIRECTLY BEFORE USE

Reusable dental tool for vertical cold condensation of gutta-percha.
 The tools are made of corrosive – resistant stainless steel.

HANDLING DENTAL TOOLS MADE OF STAINLESS STEEL

Special processes have been undertaken during the manufacture of the dental instruments to ensure corrosion-resistance. In spite of this, the durability of these dental instruments depends on their proper use and correct maintenance.

Cleaning, rinsing and washing

The packaged instruments are non – sterile. Before sterilization, these new instruments should be washed thoroughly with warm water and using detergent. Before each use these dental instruments must be sterilized.

Sterilization

All (reusable) dental instruments must be disinfected, then washed out to remove detergent and then dried before sterilization. Ultrasonic cleansing is recommended in particular as the most effective sterilization.

Recommended sterilisation methods:

Method	Recommended temperature and duration
Steam autoclave	121°C, for 20 minutes
Dry air	160°C, from 60 to 120 minutes
Chemical vapours	132°C, for appr. 20 minutes

PRECAUTIONS

Adhere to the guidelines for cleaning and sterilization of the dental instruments.

STORAGE

Dental instruments should be stored in temperatures ranging between 5-30 °C and relative humidity below 70 %. Before use protect the instrument against contamination or damage of the material structure (cracks, bends, fractures, peeling, etc.). Failure to adhere to the above stated instructions decreases instrument durability and reduces instrument lifetime.

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