PL

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INSTRUCTION FOR USE

### INTENDED USE

Protective visor reduces the risk of infections that are transferred by air and protects against the effects of lesser harmful particles during a procedure.

### METHOD OF ASSEMBLY

Remove the foil carefully, avoid contamination and scratches. Insert wider edges of the foil into the clips on the both sides of the frame. In case of the visor attached to corrective eye glasses place the foil directly on the sides of corrective eye glasses by means of the clips affixed on the foil.

Fix the foil when the frame of corrective glasses unfolded.

The foil can be individually adjusted to suit the user.

Do not fold the spectacle frame/ eye glasses with the visor foil attached; take off the foil before folding.

NOTICE: Visors are made of thermoplastic material. To shape the visor to a desired shape, soak in hot water beforehand and then shape as required.

### MAINTENANCE

Remove contaminated foil from the spectacle frame/eye glasses, wash it in warm water adding some detergent and let it dry. Don't wipe. It lets keep good transparency of Protective Visor and enables safety work. Sheet can be reused. It is recommended to disinfect the spectacle frame.

If any scratches or blurring occur as result of use, discard the foil and use another sheet of replaceable foil.

Manufacturer does not take any responsibility for using the visor in a manner not in accordance with instructions or intended use.

### CONTENTS OF SET (variants possible)

1. COMFORT VISOR – spectacle frame with metal clips  
+ set of exchangeable foils
2. COMFORT LIGHT VISOR – spectacle frame with plastic clips  
+ set of exchangeable foils
3. VISOR ATTACHED to corrective eye glasses – set of foils with attached clips for fitting to corrective eye glasses.

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