PL

ALUSTAT FOAM

Pianka do tamowania krwawienia

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ALUSTAT FOAM

Foam for staunching bleeding

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: aluminum chloride 20%, water Ph. Eur., excipients.

INTENDED USE

Product ALUSTAT FOAM is for use during dental treatment to staunch slight bleedings.

Aluminum chloride has vasoconstrictive effect on tiny blood vessels to reduce the swelling and slight bleedings.

INSTRUCTION FOR USE

Put the supplied applicator in ethanol before use, then install the applicator on the product syringe.

To stop bleeding:

Apply ALUSTAT FOAM directly on the bleeding spot. Rinse after 2 minutes.

Repeat the action if necessary.

To retract the tissues:

1. Apply ALUSTAT FOAM into gingival sulcus using the applicator supplied. Rinse after 2 minutes. or
2. Insert retraction cord into gingival sulcus and place ALUSTAT FOAM on the cord. Rinse after 2 minutes.

Product intended for multiple use except the applicator which is for single use only.

Multiple use of the applicator may cause the risk of re-infection.

CONTRAINDICATIONS

Do not use the product on patients with hypersensitivity to the product's components.

PRECAUTIONS

It may cause irritation to the mucous membrane of the oral cavity, skin and eyes.

In case of sustained irritation seek medical advice.

In case of consumption do not induce vomiting. Drink copious amount of water. Seek medical advice.

Always use disposable gloves and protective glasses while working with the product.

STORAGE

Keep in original packaging at temperature below 25 °C.

Keep out of reach of children.

Expiry date is placed on the main container.

Once the container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Syringe of 0,8 g with preparation, disposable applicators.

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