PL

# CANAL CLEAN

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EN

# CANAL CLEAN

Liquid for root canals rinsing and drying

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY**

**COMPOSITION:** isopropyl alcohol, acetone, ethyl acetate

#### INTENDED USE

Product CANAL CLEAN is intended for use during dental treatment to rinse root canal.

It accelerates drying of the root canal before filling.

**Adapter with a plug for taking the preparation into the syringe**

1. Unscrew the bottle cap.
2. Screw adapter on the bottle.
3. In order to use, remove the blue plug, screw the end of the syringe into the adapter's opening, slope the bottle with preparation upside down and take the liquid into the syringe, while holding down the adapter in the same time. Next put the bottle upright on the table top and gently unscrew the syringe from adpter.
- It is recommended to use luer-lock syringes with rubber plunger.
- 4.After each use close the adapter's opening with a blue plug.
- 5.For transportation remove adapter and twist the bottle with a bottle cap.

#### INSTRUCTIONS FOR USE

After precise preparation of the root canal, use CANAL CLEAN as the last medium; apply the product by means of the syringe with endodontic needle and afterwards dry the canal with paper points.

#### CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

#### PRECAUTIONS

It may cause skin irritation. It is irritating to the mucous membrane of the mouth. In case of contact rinse immediately with water. Product is irritating to eyes. In case of contact rinse immediately with copious amount of water and seek medical advice. In case of consumption do not extract emesis. Drink copious amount of water. Seek medical advice. It may cause sleepiness and dizziness.

Use rubber-dam while working with the product. Necessary use of disposal gloves, protective glasses and protective clothing.

**DISPENSER IS NOT INTENDED FOR THE LIQUID STORAGE!**

#### STORAGE

Store in the original packaging at temperature below 25 °C.

Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

#### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

#### PACKAGING

Bottle of 45 ml of preparation, adapter with a plug, dispenser.

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