PL

## CANAL DETECTOR

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## CANAL DETECTOR

Root canals orifices indicator

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** methylene blue, propylene glycol, water Ph. Eur.

#### INTENDED USE

Product CANAL DETECTOR is intended for use during endodontic treatment to identify the root canal orifices. Blue dye contained in the product penetrates into root canal orifices turning them blue and making it easier to find the main root canal orifices and location of any cracks.

#### INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator in spirit (ethyl alcohol) and then place it onto the syringe. Remove dentine where the root canal orifices are expected, insert 1-3 drops of preparation on the bottom of the tooth cavity, wait 5-10 seconds, next rinse and dry the tooth cavity. Points colored intensive blue indicate the root canal orifices. After product use withdraw the syringe piston to prevent overflow of preparation, screw off applicator and then screw the white plug on the syringe. Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

#### CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

#### PRECAUTIONS

Use rubber-dam while working with the product. Use disposal gloves, protective glasses and clothing.

#### STORAGE

Store in original packaging at temperature below 25° C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

#### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

#### PACKAGING

Syringe of 2 ml of preparation, disposal brush applicators .

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