PL

## Endo-Prep CREAM

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## Endo-Prep CREAM

Cream for root canals preparation

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** disodium edetate (EDTA 15%), urea peroxide 10%, excipients.

### INTENDED USE

Product ENDO-PREP CREAM is intended for chemo-mechanical root canal preparation. EDTA contained in the product helps to remove calcium salts and pulp from decalcified root canals, enabling for faster cleansing and shaping the canals. When used together with sodium hypochlorite it has whitening effect on teeth which can be discolored as a consequence of necrosis. The foam which arise as a result of the reaction between urea peroxide and NaOCl allows to remove the debris remainings and dentine from the canal after its preparation.

Thanks to the lubricant substances contained in the product, it is much easier to work with the root canal instruments and the risk of breakage of the tool in the root canal is minimized.

### INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator into the spirit (ethyl alcohol) and then place it onto the syringe. Open the tooth cavity and rinse it with sodium hypochlorite. Insert the preparation into tooth cavity by means of attached applicator or place it directly on the root canal instrument.

Prepare the canal mechanically while rinsing it with sodium hypochlorite simultaneously. Rinse the preparation out of the canals and the tooth cavity by means of sodium hypochlorite until the bubbling process is finished. Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

### CONTRAINDICATIONS

Do not use ENDO-PREP CREAM for patients with hypersensitivity to the product's components. Do not leave ENDO-PREP CREAM in the canal or in the tooth cavity between the visits. It may soften the dentin too much.

### PRECAUTIONS

Caution should be taken during canal preparation close to the apex area, because it may cause perforation of the canal as a result of use of root canal instruments. It may cause eyes and skin irritation. In case of contact rinse with copious amount of water. Seek medical advice if irritation doesn't stop. In case of consumption do not extract emesis. Drink plenty of water. Seek for medical advice. Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and clothing.

### STORAGE

Store in original packaging at temperature between 2-15° C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

### PACKAGING

Syringe of 2 ml, 5 ml or 10 ml of preparation, disposable applicators.

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