PL

ENDO-SOLution

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ENDO-SOLution

Liquid for root canals widening

PLEASE READ CAREFULLY BEFORE USE
PRODUCT FOR DENTAL USE ONLY

INGREDIENTS: disodium edetate (EDTA 17%), water Ph. Eur., excipients.

INTENDED USE

Product ENDO-SOLUTION is intended for mechanical root canal preparation. The product facilitates preparation and cleaning of the canal, removes the smear layer and exposes dentin tubule orifices. By flushing out calcium and magnesium ions, disodium edetate (EDTA) softens dentine surface layer to allow its easier removal and restoration of the canal.

Adapter with a cup for drawing the preparation into the syringe

- Unscrew the bottle cup.
- Screw adapter on the bottle.
- Remove the blue plug, screw the end of the syringe into the opening of the adapter, tilt the bottle with preparation upside down and draw the liquid into the syringe, while simultaneously holding the adapter. Next put the bottle upright on the table top and gently unscrew the syringe from adapter. Luer-lock syringers with rubber plunger are recommended.
- Screw on the blue cap after each use.
- For shipping, remove the adapter and screw the cap on.

INSTRUCTIONS FOR USE

To prepare the root canal insert 2-3 drops of liquid ENDO-SOLUTION into the canal simultaneously preparing it mechanically for approx. 30 seconds. Rinse it with physiological saline. Repeat the process a few times. Rinsing with sodium hypochloride (CHLORAXID) is recommended.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product components. Do not leave product in the canal or in the tooth cavity between the visits. It may soften the dentin too much.

PRECAUTIONS

Special care should be taken when product applied into perforations. It may irritate the skin and eyes. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, drink copious amount of water. Do not induce vomiting. Seek medical advice. Use rubber-dam when working with the product. Use gloves, protective glasses and clothing. **DISPENSER IS NOT INTENDED FOR THE LIQUID STORAGE!**

STORAGE

Store in original packaging below 25° C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Bottle contains 50 ml, 120 ml or 200 g of preparation, adapter with a cap, dispenser.

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