**PL**

# ENDO-SOLution

## premium

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## premium

Liquid for root canals widening

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY**

**COMPOSITION:** disodium edetate (EDTA 17%), surfactants, water Ph. Eur., excipients.

### INTENDED USE

Product ENDO-SOLUTION PREMIUM is intended for mechanical root canals preparation. Disodium edetate (EDTA) contained in the product by rinsing calcium and magnesium ions softens the surface layer of the dentine and makes it easier to remove it and restore the root canal.

The product contains surfactants which improve penetration of the preparation and accelerates removing the smear layer from the root canal, removing at the same time root canal orifices.

**Adapter with a plug for taking the preparation into the syringe**

1. Unscrew the bottle cap.
2. Screw adapter on the bottle.
3. In order to use, remove the blue plug, screw the end of the syringe into the adapter's opening, slope the bottle with preparation upside down and take the liquid into the syringe, while holding down the adapter in the same time. Next put the bottle upright on the table top and gently unscrew the syringe from adapter. It is recommended to use luer-lock syringes with rubber plunger.
4. After each use close the adapter's opening with a blue plug.
5. For transportation remove adapter and twist the bottle with a bottle cap.

### INSTRUCTIONS FOR USE

To restore the root canal insert 2-3 drops of liquid ENDO-SOLUTION PREMIUM into the canal preparing it at the same time mechanically for about 30 seconds. Rinse it with physiological saline. Repeat the process a few times. It is recommended as well to rinse root canals with sodium hypochlorite (CHLORAXID).

### CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components. Do not leave product in the canal or in the chamber of the tooth between visits. This may result in excessive softening of dentine.

### PRECAUTIONS

Special care should be taken when product applied into perforations. It may be irritating to skin and eyes. In case of contact, rinse immediately with plenty of water and seek for medical advice. In case of consumption drink copious amount of water. Do not extract emesis. Seek for medical advice. Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and clothing. **DISPENSER IS NOT INTENDED FOR THE LIQUID STORAGE!**

### STORAGE

Store in original packaging at temperature below 25° C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

### PACKAGING

Bottle of 120 ml of preparation, adapter with a plug, dispenser.

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