PL

Eucalyptol

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Eucalyptol

Oil for dissolving gutta-percha

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: eucalyptus oil 100%

INTENDED USE

Product EUCALYPTOL is intended for use during endodontic re-treatment to remove gutta-percha from the root canal.

INSTRUCTIONS FOR USE

Before the treatment immerse attached pipette into the spirit (ethyl alcohol)

- Insert 1-2 drops of liquid into the root canal by means of attached pipette.
- Remove gutta-percha.
- Use another dose of preparation if necessary.
- If gutta-percha is still dry or fragile, apply EUCALYPTOL again.
- Repeat the action as long as needed.

Product intended for multiple use except the pipette which is for single use only.

Multiple use of pipette may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS

Use rubber-dam, gloves, protective glasses and protective clothing while working with the product.

STORAGE

Store in original packaging at temperature below 25 °C.

Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned back to the manufacturer.

PACKAGING

Bottle of 10 ml of preparation, application pipettes.

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