PL

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EN

FOG BUSTER for Protective Visor  
Anti-Fog Spray. Removes dirt

**PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE**  
PRODUCT FOR DENTAL USE ONLY

**INGREDIENTS:** water, sodium lauryl sulfate, excipients.

#### INDICATIONS FOR USE

The Fog Buster spray is intended for use with the Protective Visor.  
Fog Buster prevents deposition of vapour on surface of the Protective Visor and removes dirt.  
The spray can also be used for protection of other surfaces that may be exposed to deposition of vapour.

#### INSTRUCTIONS FOR USE

Shake before use. Spray evenly on the Protective Visor film surface. Do not rinse. Wipe with a soft cloth. Wash hands after use.

#### CONTRAINDICATIONS

Do not spray into eyes.  
Do not ingest.

#### PRECAUTIONS

This product can cause eye irritation. In case of contact with eyes, rinse with plenty of water and contact a doctor immediately.  
If swallowed, drink plenty of water. Contact a doctor immediately.

#### STORAGE

Store in a tightly closed original container below 25°C.  
Keep out of the reach of children.  
The expiry date is on the direct packaging.

#### DISPOSAL OF PACKAGING

Waste containers should be disposed of or returned to the manufacturer.

#### PACKAGING

Atomizer bottle containing 30 ml of the product.

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