PL

## GLUCO-CHEX 2% gel

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## GLUCO-CHEX 2% gel

Gel for root canals preparation

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** chlorhexidine digluconate 2%, base gel.

### INTENDED USE

Product GLUCO-CHEX GEL 2% is intended for use during endodontic treatment, it facilitates mechanical root canal preparation. Product lubricates a root canal during its mechanical preparation, lubricants contained in the product enable easy introduction of an instrument into a canal and the risk of its breakage is minimized.

### INSTRUCTIONS FOR USE

Before the treatment immerse the attached applicator into the spirit (ethyl alcohol) and then place it onto the syringe. Introduce the product into the root canal directly on the canal instrument or by means of the attached applicator, at the same time prepare the canal mechanically. While preparing the canal rinse it with physiological saline or distilled water. Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

### CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components. **CAUTION!** Do not rinse with sodium hypochlorite since it may cause precipitation of a tawny deposit which is hard to remove.

### PRECAUTIONS

Caution should be taken during canal preparation close to the apex area because it may cause perforation of the canal as a result of use of root canal instruments. It may cause irritation to the mucous membrane of the oral cavity and eyes. In case of contact rinse immediately with copious amount of water. In case of consumption do not induce vomiting. Drink copious amount of water. Seek medical advice.

### STORAGE

Store out of reach of children. Protect from direct sunlight. Store in original packaging at temperature below 25 °C. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

### PACKAGING

Syringe of 5 ml or 10 ml, dispenser, connector, disposal applicators.

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