PL

# GLUCOSITE gel

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# GLUCOSITE gel

Gel for gingival pockets

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY****INGREDIENTS:** water, methylcellulose, glycerine, ethyl alcohol, chlorhexidine digluconate, hydrogen peroxide**INTENDED USE**

Product GLUCOSITE GEL is intended for use during dental treatment, by the dentist, for rinsing the gingival pockets and other difficult to be reached areas of mucosa membrane of the oral cavity in purpose of washing and cleaning them.

**INSTRUCTIONS FOR USE**

Put the supplied applicator in ethanol before use, then install the applicator on the product syringe. By means of attached applicator insert gel into the gingival pocket, apply preparation into the pocket by shifting applicator along the gingiva's line. When application is finished it is recommend to rinse the oral cavity with clean water. If necessary repeat the treatment after about 7 days.

**CONTRAINDICATIONS**

Do not use the product for people with hypersensitivity to the product's components.

**CAUTIONS**

In case of contact with eyes, rinse immediately with copious amount of water and seek medical advice. In case of consumption do not extract emesis. Drink copious amount of water. Seek medical advice.

**STORAGE**

Store in original packaging at temperature below 25 °C. Store out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date doesn't change if it is securely closed after each use.

**HANDLING THE PRODUCT CONTAINER**

Empty containers should be disposed of responsibly or returned to the manufacturer.

**PACKAGING**

Syringe of 2 ml of preparation, disposal applicators. Bottle of 50 ml of preparation, adapter with plug, dispenser, disposable applicators.

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