

PL

FOLIA WYMIENNA SMART

INSTRUKCJA UŻYCIA

PRZEZNACZENIE

Przeznaczona do używania z oprawkami okularowymi Przyłbicy Ochronnej CERKAMED.

SPOSÓB MOCOWANIA

Delikatnie wyjąć folię, unikając zabrudzenia i zarysowania. Umieścić końce arkusza folii w uchwytach po bokach oprawki okularowej.

Mocować folię, gdy oprawka jest rozłożona.

Wysokość oraz odchylenie folii można regulować indywidualnie.

Nie należy składać oprawki z zamocowaną folią, przed złożeniem folię należy wypiąć.

KONSERWACJA

Po zabrudzeniu wypiąć folię z oprawki, myć w ciepłej wodzie z dodatkiem detergentu, spłukać i pozostawić do wyschnięcia. Nie wycierać. Pozwala to na zachowanie dobrej przejrzystości folii.

Folię można użyć ponownie. W przypadku pojawienia się na folii zarysowań lub zmatowienia, będącego wynikiem zużycia – folię należy wyrzucić i użyć kolejnego arkusza.

Producent nie ponosi odpowiedzialności za konsekwencje użycia wyrobu niezgodnie z instrukcją i przeznaczeniem.

OPAKOWANIE ZAWIERA

4 sztuki folii.

Oprawka okularowa nie wchodzi w skład zestawu.

EN

EXCHANGEABLE FOILS SMART

INSTRUCTION FOR USE

INTENDED USE

The foils are suitable for use with Protective Visors CERKAMED.

METHOD OF ASSEMBLY

Remove the foil carefully, avoid contamination and scratches. Insert wider edges of the foil into the clips on the both sides of the frame. Fix the foil when the frame of corrective glasses unfolded.

The foil can be individually adjusted to suit the user.

Do not fold the spectacle frame with the visor foil attached; take off the foil before folding.

MAINTENANCE

Remove contaminated foil from the spectacle frame, wash it in warm water adding some detergent and let it dry. Don't wipe. High breaking resistance of the foils.

Sheet can be reused. If any scratches or blurring occur as result of use, discard the foil and use another sheet of replaceable foil.

Manufacturer does not take any responsibility for using the visor in a manner not in accordance with instructions or intended use.

CONTENTS OF SET

4 pcs of foils.

Glasses are not included in the set.



PPH CERKAMED Wojciech Pawłowski
37-450 Stalowa Wola, Poland
ul. Kwiatkowskiego 1
Tel/fax +48 15 842 35 85

Data ostatniej aktualizacji/Last update: 27.07.2020/1