PL

CANAL CLEAN

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EN

CANAL CLEAN

Liquid for root canals rinsing and drying

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY

COMPOSITION: isopropyl alcohol, acetone, ethyl acetate.

INTENDED USE

CANAL CLEAN is intended for use during dental treatment to rinse root canals. It accelerates drying of the root canal before filling.

Adapter with a plug for taking the liquid preparations into the syringe

1. Unscrew the bottle cap.
2. Screw the adapter on the bottle.
3. To use, remove the blue plug, screw the end of the syringe into the opening of the adapter, tilt the bottle upside down and pull the liquid into the syringe, while simultaneously holding down the adapter. Next, put the bottle upright on the table top and gently unscrew the syringe from the adapter. Use luer-lock syringes with a rubber plunger.
4. After each use, close the adapter's opening with a blue plug.
5. For transport, remove the adapter and close the bottle with a bottle cap.

INSTRUCTIONS FOR USE

After thorough preparation of the root canal, use CANAL CLEAN as the last medium; apply the product by using a syringe with an endodontic needle and afterwards dry the canal with paper points.

CONTRAINDICATIONS

Do not use for patients who are hypersensitive to the product components.

PRECAUTIONS

It may cause skin irritation. Irritating to the mucous membrane of the mouth. In case of contact, rinse immediately with water. Eye irritant. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting. Drink plenty of water.

Seek medical advice. It may cause sleepiness and dizziness.

Use a rubber-dam when working with the product.

Use protective gloves, glasses and protective clothing.

DISPENSER IS NOT INTENDED FOR STORING LIQUIDS!

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Keep in original packaging below 25 °C.

Keep out of reach of children.

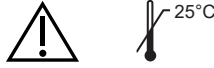
Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Bottle with 45 ml of the preparation, adapter with a plug, dispenser.

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