PL

PROTECT LIGHT SEAL

Lakier ochronny do zębiny

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PROTECT LIGHT SEAL

Protective dentine enamel

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY

COMPOSITION: 1% potassium fluoride (3500 ppm F), HEMA, Ph Eur. purified water, excipients.

INTENDED USE

- Dentine hypersensitivity treatment and prevention especially in the area of the tooth neck
- Base for cementation or reconstruction of bone tissue
- Post-treatment hypersensitivity
- Before taking impressions (on patients with sensitivity to cold)
- After descaling treatment
- Liner for restorative materials

PROTECT LIGHT SEAL is a protective varnish for applying on the dentine, intended for use during dental treatment in order to reduce hypersensitivity and on the exposed teeth necks.

It improves remineralisation of the dentine and due to fluoride content, supports closing the dentinal tubules. It leaves a protective layer on the dentine surface.

INSTRUCTIONS FOR USE

Apply the varnish on the tooth surface with a disposal brush applicator, wait for 30 seconds and dry with a gentle stream of air.

The varnish does not require light curing. When applying a few layers of the preparation, the usage of polymerization lamp hastens curing process of the varnish. The brush applicator is for single use only. Multiple use of the brush applicator may cause the risk of reinfection.

CONTRAINDICATIONS

Do not use in patients who are hypersensitive to any of the ingredients. Do not use in case of pulp inflammation in dentine area.

PRECAUTIONS

Use only when inflammation has been successfully treated.

While working with the product, use rubber dam, gloves, protective glasses and clothing. The product is sensitizing. May cause eyes and skin irritation. It may cause allergies. In case of contact, immediately rinse with plenty of water. Swallowing large amounts of product may be harmful. In case of consumption do not induce vomiting, drink plenty of water and contact a doctor. CAUTION: Highly inflammable. Contains acetone. Keep away from fire. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in a tightly closed original container below 25 °C. Protect against direct sunlight. Close tight after use. Keep out of reach of children. The expiry date is on the direct packaging. CAUTION: Once the container is open, the product keeps its properties within 6 months.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Vial with 1 ml of preparation, disposable brush applicators.

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