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EN
<b>PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE PRODUCT FOR DENTAL USE ONLY</b>
COMPOSITION: o-phosphoric acid 36%, gel base, dye.
INTENDED USE BLUE ETCH FLOW is used for etching dentine and enamel fissures prior to composites and cavity seals. With its carefully formulated semi-solid consistence, BLUE ETCH FLOW reaches inaccessible tight grooves and fissures. The product is also recommended for use before cementing glass fibre crown and root inlays.
PREPARATION OF THE PRODUCT BEFORE USE (for 10 ml and 50 ml variants) 1. Unscrew the white plug from the dosing syringe containing etchant. 2. Replace the white plug with the attached connector. 3. Unscrew the blue plug from the connector and replace it with dispenser – a smaller syringe attached to the product. 4. Push down the plunger of dosing syringe and pump the required amount of gel into the dispenser, but no more than 2 ml. 5. Unscrew the filled dispenser and screw the attached applicator onto the top (application needle with a blunt end). The product is ready for use. 6. Leave the connector screwed on the dosing container and protect it with the blue plug.
INSTRUCTIONS FOR USE Before the treatment, immerse the attached applicator into ethyl alcohol and then place it onto the syringe. Apply a thin layer of the etchant onto the dried surface. Etch no longer than 60 seconds. Irrigate with a strong air stream and water (about 15-20 seconds). Do not over-dry. In case of enamel impurities, repeat the process. A reusable product with a single-use applicator. Re-using the applicator may cause the risk of reinfection.
CONTRAINDICATIONS Do not use the product for patients with hypersensitivity to the product components.
PRECAUTIONS May cause severe skin burns and eye damage. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting. Drink plenty of water. Seek medical advice. Use a rubber dam when working with the product. Use protective gloves, glasses and protective clothing. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.
STORAGE Store in original packaging below 25°C. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.
HANDLING THE PRODUCT CONTAINER Empty containers should be disposed of responsibly or returned to the manufacturer.
PACKAGING Syringe with 2 ml of preparation, disposable applicator. Syringe with 10 ml or 50 ml with preparation, disposable applicators, connector and dispenser.
Last update: 15.04.2021/4

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EN
<b>PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE PRODUCT FOR DENTAL USE ONLY</b>
COMPOSITION: o-phosphoric acid 36%, gel base, dye.
INTENDED USE BLUE ETCH FLOW is used for etching dentine and enamel fissures prior to composites and cavity seals. With its carefully formulated semi-solid consistence, BLUE ETCH FLOW reaches inaccessible tight grooves and fissures. The product is also recommended for use before cementing glass fibre crown and root inlays.
PREPARATION OF THE PRODUCT BEFORE USE (for 10 ml and 50 ml variants) 1. Unscrew the white plug from the dosing syringe containing etchant. 2. Replace the white plug with the attached connector. 3. Unscrew the blue plug from the connector and place it with dispenser – a smaller syringe attached to the product. 4. Push down the plunger of dosing syringe and pump the required amount of gel into the dispenser, but no more than 2 ml. 5. Unscrew the filled dispenser and screw the attached applicator onto the top (application needle with a blunt end). The product is ready for use. 6. Leave the connector screwed on the dosing container and protect it with the blue plug.
INSTRUCTIONS FOR USE Before the treatment, immerse the attached applicator into ethyl alcohol and then place it onto the syringe. Apply a thin layer of the etchant onto the dried surface. Etch no longer than 60 seconds. Irrigate with a strong air stream and water (about 15-20 seconds). Do not over-dry. In case of enamel impurities, repeat the process. A reusable product with a single-use applicator. Re-using the applicator may cause the risk of reinfection.
CONTRAINDICATIONS Do not use the product for patients with hypersensitivity to the product components.
PRECAUTIONS May cause severe skin burns and eye damage. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting. Drink plenty of water. Seek medical advice. Use a rubber dam when working with the product. Use protective gloves, glasses and protective clothing. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.
STORAGE Store in original packaging below 25°C. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.
HANDLING THE PRODUCT CONTAINER Empty containers should be disposed of responsibly or returned to the manufacturer.
PACKAGING Syringe with 2 ml of preparation, disposable applicator. Syringe with 10 ml or 50 ml with preparation, disposable applicators, connector and dispenser.
Last update: 15.04.2021/4

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