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**EN**
**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**
**PRODUCT FOR DENTAL USE ONLY**

**COMPOSITION:** sodium hypochlorite (5.25% of active chlorine), water Ph. Eur., surfactants.

**INTENDED USE**
Liquid for root canal rinsing. Removal of dead pulp residues during mechanical preparation of the root canal. Cleans the root canal, including the smear layer to expose the dental tubule orifices prior to filling the canal. Prevents discoloration of the teeth that might occur after filling the non-rinsed root canal.

**CONTRA-INDICATIONS**
Do not use for patients who are hypersensitive to the product components.
**CAUTION!** Flushing root canals: separate the sodium hypochlorite flush and the chlorhexidine digluconate flush with a normal saline flush to prevent a brownish sediment from forming. Thoroughly flush and rinse.
**PRECAUTIONS**
During root canal preparation do not use a too heavy stream of the liquid because it can cause the liquid to penetrate the periapical tissues and damage them, resulting in pain and swelling of soft tissues. Special care should be taken when the product is applied into perforations. May cause severe skin burns and eye damage. In case of contact rinse immediately with plenty of water and seek medical advice. In case of consumption do not induce vomiting. Drink plenty of water. Seek medical advice. Use rubber-dam while working with the product. The use of personal protective equipment (glasses, protection visor, gloves) is recommended.
**DISPENSER IS NOT INTENDED FOR LIQUID STORAGE!**
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

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**STORAGE**
Store in original containers at 2-8°C (refrigerate). Protect from sunlight. Keep out of reach of children.
Once container is open, the expiry date does not change if the container is securely closed after each use.

**PRECAUTIONS**
During root canal preparation do not use a too heavy stream of the liquid because it can cause the liquid to penetrate the periapical tissues and damage them, resulting in pain and swelling of soft tissues. Special care should be taken when the product is applied into perforations. May cause severe skin burns and eye damage. In case of contact rinse immediately with plenty of water and seek medical advice. In case of consumption do not induce vomiting. Drink plenty of water. Seek medical advice. Use rubber-dam while working with the product. The use of personal protective equipment (glasses, protection visor, gloves) is recommended.
**DISPENSER IS NOT INTENDED FOR LIQUID STORAGE!**
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

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