

# RUBBER-DAM CORD

Gumka zaciskowa do koferdamu

**PRZED UŻYCIEM NALEŻY ZAPOZNAĆ SIĘ Z PONIŻSZĄ INSTRUKCJĄ**  
WYRÓB PRZEZNACZONY DO STOSOWANIA WYŁĄCZNIE PRZEZ LEKARZA STOMATOLOGA

## ZASTOSOWANIE

W celu ustabilizowania koferdamu umieścić gumkę w przestrzeni międzyzębowej pomiędzy zębem odśloniętym, a zębem zabezpieczonym koferdamem.

Do ustabilizowania koferdamu wystarczy gumka o długości około 4 cm. Po zakotwiczeniu długość gumki może być dostosowana do potrzeb, poprzez odcięcie nożyczkami.

Użycie lubrykantu rozpuszczalnego w wodzie, ułatwi zamocowanie gumki w przestrzeni międzyzębowej.

## PRZECHOWYWANIE

Przechowywać w oryginalnym opakowaniu w temperaturze do 25°C.

Chronić przed promieniowaniem słonecznym.

Szczelnie zamknąć po użyciu.

Przechowywać w miejscu niedostępnym dla dzieci.

Termin przydatności do użycia znajduje się na opakowaniu bezpośrednim.

## OSTRZEŻENIA

Produkt zawiera latex, co może wywoływać reakcję uczuleniową.


## POSTĘPOWANIE Z OPAKOWANIAM I PO WYROBIE

Zużyte opakowania należy przekazać do utylizacji lub zwrócić do wytwórcy.

## ZAWARTOŚĆ

Dozownik zawierający 1,2 m gumki.

Dostępne warianty grubości: cienka, średnia i gruba.

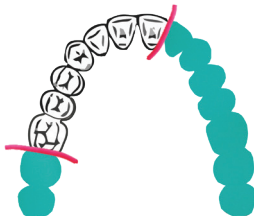

PPH CERKAMED Wojciech Pawłowski

37-450 Stalowa Wola, Polska

ul. Kwiatkowskiego 1

Tel/fax 0048 15 842-35-85

# RUBBER-DAM CORD

Dental dam stabilizing cord

**READ THE FOLLOWING INSTRUCTION BEFORE USE**  
PRODUCT FOR DENTAL USE ONLY

## USE

To stabilize dental dam place the cord in the interdental spaces between the exposed tooth and the one protected with dam.

4 cm long of cord is sufficient to stabilize the dental dam. After placement, the length of the cord can be adjusted by snipping with scissors.

The use of watersoluble lubricant facilitates the cord fixing in interdental spaces.

## STORAGE

Keep in the original packaging, below 25°C.

Do not expose to solar radiation.

Close tight after the use.

Keep out of reach of children

The expiry date is printed on the direct package.

## PRECAUTIONS

This product contains latex which may cause allergic reactions.


## PACKAGE WASTE DISPOSAL

Used packaging must be disposed of responsibly or returned to the manufacturer.

## PACKAGING

Dispenser containing 1,2 m of cord.

Available sizes: fine, medium, thick.

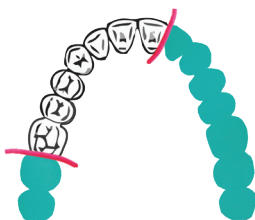

PPH CERKAMED Wojciech Pawłowski  
37-450 Stalowa Wola, Polska  
ul. Kwiatkowskiego 1  
Tel/fax 0048 15 842-35-85