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EN
<p>BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY</p> <p>PRODUCT FOR DENTAL USE ONLY</p> <p>NON - STERILE PRODUCT. STERILIZE DIRECTLY BEFORE USE</p>

The tools are made of corrosive – resistant stainless steel.

COMPONENTS
Product contains:

- HANDLE
- PISTON

The piston is used to squeeze the cement to the treatment area.

HANDLING DENTAL TOOLS MADE OF STAINLESS STEEL
Special processes have been undertaken during the manufacture of the dental instruments to ensure corrosion-resistance. In spite of this, the durability of the dental instruments depends on their proper use and correct maintenance.

CLEANING, RINSING AND WASHING

The packaged instruments are non – sterile. Before sterilization, these new instruments should be washed thoroughly with warm water and using detergent. Before each use these dental instruments must be sterilized.

STERILIZATION

All (reusable) dental instruments must be disinfected, then washed out to remove detergent and then dried before sterilization. Before use protect the instrument against contamination or damage of the material structure (cracks, bends, fractures, peeling, etc.) Failure to adhere to the above stated instructions decreases instrument durability and reduces instrument lifetime.

Recommended sterilisation methods:

Method	Minimal parameters:
Steam autoclave	134°C, for 5 minutes

PRECAUTIONS
Adhere to the guidelines for cleaning and sterilization of the dental instruments.

STORAGE

Dental instruments should be stored in temperatures ranging between 5-30°C and relative humidity below 70%.
Before use protect the instrument against contamination or damage of the material structure (cracks, bends, fractures, peeling, etc.) Failure to adhere to the above stated instructions decreases instrument durability and reduces instrument lifetime.

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