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**EN**

<b>BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY</b> <b>PRODUCT FOR DENTAL USE ONLY</b>
<b>COMPOSITION:</b> methacrylic silane, organic solvent.
<b>INTENDED USE</b> SILANE is applied on porcelain after etching, before using bonding system in order to increase the durability of bonding between resin and porcelain. The product can be used for silanizing the glass fibres.
<b>INSTRUCTIONS FOR USE</b> Before the treatment immerse attached applicator in alcohol (ethyl alcohol) and then fix it onto the syringe.
1. Screw the brush applicator onto the syringe with SILANE, replacing the closing plug with the applicator.
2. Rub SILANE into previously etched porcelain surface.
3. Do not rinse! Use air flow to dry.
4. Porcelain surface is ready for bonding system application.
Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.
<b>CONTRAINDICATIONS</b> Do not use the product for patients with hypersensitivity to the product’s components.
<b>PRECAUTIONS</b> Use a rubber dam and protection for patient’s eyes when repairing teeth in the patient’s oral cavity. Use a vacuum system while etching. Equip staff with protective clothing, gloves and visors. The product may irritate the skin, eyes and the respiratory system. In case of eye contact, rinse with plenty of water and contact a doctor. In case of contact with the skin, wash it out with soap and plenty of water. If swallowed, do not induce vomiting. Rinse mouth with water. Contact a doctor.
<b>STORAGE</b> Store in original packaging below 25°C. Keep out of reach of children. The expiry date is not the direct packaging. Once container is open, the expiry date doesn’t change if the container is securely closed after each use.
<b>HANDLING THE PRODUCT CONTAINER</b> Empty containers should be disposed of responsibly or returned to the manufacturer.
<b>PACKAGING</b> A syringe containing 2 ml of the formulation.
Last update: 24.05.2021/4

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