PL **GLUCOSITE GEL**

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EN **GLUCOSITE GEL**

Gel for gingival pockets

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

INGREDIENTS: water, methylcellulose, glycerin, ethyl alcohol, chlorhexidine digluconate, hydrogen peroxide

INTENDED USE

GLUCOSITE GEL is intended for use during dental procedures carried out by a doctor, for irrigating gingival pockets and other difficult to reach areas of the oral mucosa to wash and cleanse.

INSTRUCTIONS FOR USE

Put the supplied applicator in ethanol before use, then install the applicator on the syringe. Using the included applicator, insert the gel in the gingival pocket and continue moving the applicator along the gum line. When the application is finished, rinse the mouth with clean water. If necessary, repeat the treatment after about 7 days.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

PRECAUTIONS

In case of contact with eyes, rinse immediately with copious amount of water and seek medical advice.

In case of consumption seek medical advice.

STORAGE

Store in original packaging below 25 °C.

Keep out of reach of children. Expiry date is placed on the main container.

Once the container is open, the expiry date doesn't change if it the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Syringe containing 2 ml of the product, disposable applicators. Bottle containing 50 ml of product, adapter with screw cap, dispenser, disposable applicators.

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