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EN

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: sodium hypochlorite (5,25% of active chlorine), water Ph. Eur.

INTENDED USE
Liquid for root canal rinsing. Removal of dead pulp residues during mechanical preparation of the root canal. It cleans the canal and removes the smear layer to expose the dentin root canal orifices before final filling.
It prevents discoloration of the teeth that might occur after filling the non-rinsed root canal.

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INSTRUCTIONS FOR USE
Insert CHLORAXID 5,25% into the root canal using the syringe with a needle with a side opening and start rinsing.
Sodium hypochlorite in the product inactivates immediately in contact with organic substances. Therefore, to compensate, the amount of the preparation should be increased to about 10 ml per canal.

CONTRAINDICATIONS
Do not use the product CHLORAXID 5,25% for patients with hypersensitivity to the product's components.

CAUTION! While rinsing root canals, use physiological saline between rinsing with sodium hypochlorite and chlorhexidine digluconate. It prevents the precipitation of a brown deposit.
Rinse copiously.

PRECAUTIONS
During canal preparation do not use a too heavy stream of the liquid because it can cause the liquid to penetrate the periapical tissues and damage them, resulting in pain and swelling of soft tissues.
Special care should be taken when the product is applied into perforations. May cause severe skin burns and eye damage.
In case of contact rinse immediately with plenty of water and seek medical advice. In case of consumption do not induce vomiting. Drink plenty of water. Seek medical advice. Use rubber-dam while working with the product. The use of personal protective equipment (glasses, protective visor, gloves) is recommended.
DISPENSER IS NOT INTENDED FOR LIQUID STORAGE!
Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the State in which the user or patient is domiciled.

STORAGE
Store in original packaging at 2-8°C (in a refrigerator). Store in a securely closed packaging. Protect from sunlight. Keep out of reach of children. Expiry date is placed on the main container.
In case of incident grave lié au produit, le signaler au fabricant et à l'autorité publique territorialement compétente selon le lieu d'habitation de l'utilisateur/ du patient.

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TRANSPORT
The manufacturer allows transport of the product for a period not longer than 10 days at a temperature not exceeding 40°C. Short-term transport within the recommended range does not affect the safety and efficacy of the product throughout its shelf life.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer

PACKAGING
Bottle of 200 g or 400 g of preparation, adapter with a screw cap, dispenser.
Last update: 09.08.2021/11

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