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EN
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY

COMPOSITION: zinc oxide / en. Ph. purity, excipients.

INTENDED USE
ZINC OXIDE FAST is recommended for making pastes, mainly with Eugenol, used for temporary filling of cavities and root canals.

Mixture of zinc oxide with eugenol as a temporary filling has drying effect and stimulates creation of the secondary dentine.

INSTRUCTIONS FOR USE
ZINC OXIDE FAST and eugenol should be combined in quantities allowing obtaining a paste of the desired consistency. The recommended ratio is one teaspoon of ZINC OXIDE FAST and one drop of eugenol (weight ratio 3:1) using the accessories shipped with CERKAMED products - EUGENOL and ZINC OXIDE FAST.

Leave the filling in the cavity or in the root canal for a period of 1 week but no longer than 30 days.

CAUTION!
ZINC OXIDE FAST is a fast-setting product. Mix for no longer than 30-50 seconds. Bonding time: up to 10 minutes.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product components.

PRECAUTIONS

May cause eye irritation In case of contact, rinse immediately with plenty of water and seek medical advice.
If swallowed, drink plenty of water. Do not induce vomiting. Seek medical advice. Use a rubber-dam when working with the product.
Use protective gloves, glasses and protective clothing.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in original packaging below 25°C. Protect from moisture. Keep out of reach of children.
Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Container with 50 g of the preparation.

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