PL

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EN

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY

COMPOSITION: sodium hypochlorite (5.25% of active chlorine), Ph. Eur. purified water, gel base.

INTENDED USE

Preparation for root canal rinsing. Removal of dead pulp residues during mechanical preparation of the root canal. Cleans the root canal, including the syvnovial layer to expose the dental tubule orifices prior to filling the canal. Prevents discoloration of the teeth that might occur after filling the root canal. The gel consistency of the preparation minimises the risk of pushing the product beyond the apical opening and ensures a precise control during application. CHLORAXID 5,25% GEL is specifically recommended for preparing root canals with wide, underdeveloped or damaged apexes.

INSTRUCTIONS FOR USE
Immers the supplied applicator in ethanol before use. When install the applicator on the product syringe. Prepare the root canal by pushing CHLORAXID 5,25% GEL directly into the canal with the supplied applicator or coat the intracanal instrument of choice with the product.

Prepare the root canal and flush out with normal saline using a needle with an electronic opening. Flush alternately with EDTA. Repeat several times. A reusable product with a single-use applicator. Re-using the applicator may cause the risk of reinfection.

CONTRAINDICATIONS
DO NOT USE for patients who are hypersensitive to the product components. **CAUTION!** Flushing root canals: separate the sodium hypochlorite flush and the chlorhexidine digluconate flush with a normal saline flush to prevent a brownish sediment from forming. Thoroughly flush and rinse.

PRECAUTIONS
Extreme caution is advised when working with perforations. May cause severe skin burns and eye damage. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting.

CONTRAINDICATIONS
DO NOT USE for patients who are hypersensitive to the product components. **CAUTION!** Flushing root canals: separate the sodium hypochlorite flush and the chlorhexidine digluconate flush with a normal saline flush to prevent a brownish sediment from forming. Thoroughly flush and rinse.

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