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EN
----

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**
**PRODUCT FOR DENTAL USE ONLY**

**COMPOSITION:** eugenol oil 100%

**INTENDED USE**

EUGENOL is used as a liquid for making pastes for root canal filling and temporary filling of cavities. It is mainly used with zinc oxide.

**INSTRUCTIONS FOR USE**

EUGENOL and zinc oxide should be combined in proportions allowing obtaining a paste of the desired consistency. The recommended ratio is one teaspoon of zinc oxide and one drop of EUGENOL (weight ratio 3:1) using the accessories shipped with CERKAMED products – EUGENOL and ZINC OXIDE.

**CONTRAINDICATIONS**

Do not use the product for patients with hypersensitivity to the product's components. Do not use under composites since they may disturb polymerization process. Do not use in pregnant women.

**PRECAUTIONS**

It can cause allergic skin reaction. It is irritating to the mucous membranes of the oral cavity and eyes. In case of contact rinse immediately with plenty of water and seek medical advice. In case of consumption, drink plenty of water and do not induce vomiting. Seek medical advice. Use rubber dam while working with the product. Use protective gloves, glasses and clothing. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

**STORAGE**

Store in original packaging at a temperature below 25°C. Protect from sunlight. Keep out of reach of children. Expiry date is placed on the main container. Once the container is open, the expiry date doesn't change if it is. The container is securely closed after each use.

**HANDLING THE PRODUCT CONTAINER**

Empty containers should be disposed of responsibly or returned to the manufacturer.

**PACKAGING**

Bottle of 10 ml or 20 ml of preparation with a dropper.

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