PL

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EN

PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: 40% citric acid in an aqueous solution.

INTENDED USE

The preparation is used during endodontic treatment for rinsing root canals. It improves mechanical cleaning of the root canal.

It removes smear layer from the walls of root canal exposing the dentinal tubules and allows for precise cleansing of dentinal tubules from pulp debris and precise sealant penetration. CITRIC ACID 40% is non-toxic for the human body. It doesn't pose any ecological threat.

Adapter with a plug for taking the liquid preparations into the syringe

1. Unscrew the bottle cap.
2. Screw the adapter on the bottle.
3. In order to use the bottle, remove the blue plug, screw the end of the syringe into the adapter opening, tilt the bottle with the preparation upside down and take the liquid into the syringe, while simultaneously holding down the adapter.
Next, put the bottle upright on the table top and gently unscrew the filled syringe from the adapter. It is recommended to use luer-lock syringes with a rubber plunger.
4. After each use, close the adapter opening with a blue plug.
5. For transport, remove the adapter and screw the cap on the bottle.

INSTRUCTIONS FOR USE

Insert the product into the root canal using the syringe with a needle with a side opening while mechanically preparing the canal. After using the product, rinse the canal with physiological saline.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components. Do not leave the product in the canal or the tooth cavity between visits. This may result in excessive softening of dentin.

PRECAUTIONS

Special care should be taken when the product is applied into perforations. It may irritate the eyes in case of contact with eyes.
Use caution when working with the eyed lid held open. Use rubber-dam while working with the product. Use protective gloves, glasses and clothing.

Take special care when using this product to children because of the danger of excessive softening of too hard tissues.
The product is fully biodegradable.

DISPENSER IS NOT INTENDED FOR LIQUID STORAGE!

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in closed original packaging below 25°C. Keep out of reach of children.

SHAKE BEFORE USE!

Expiry date is placed on the main container.

Once the container is open, the expiry date doesn't change if the container is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Bottle of 200 g or 400 g of preparation, adapter with a screw cap, dispenser.

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