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EN
<b>BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY</b> <p>PRODUCT FOR DENTAL USE ONLY</p>

**COMPOSITION:** 4-ethylro rhodamine, propylene glycol, surfactants.

**INTENDED USE**

RED DETECTOR is intended for locating caries lesions. Red dye in the product permeates the tissue with caries lesions and dyes it red. RED DETECTOR increases safety of work by locating the carious dentine and saving the healthy tissues.

**INSTRUCTIONS FOR USE**

Before the treatment immerse attached applicator in alcohol (ethyl alcohol) and then fix it onto the syringe. After removing all visible carious lesions, apply one drop of preparation on the loss area, wait 5-10 seconds, then irrigate and dry the cavity. Points dyed bright red indicate carious dentine. After using the product, withdraw the syringe piston to prevent overflow of preparation, screw off the applicator and then screw the white plug on the syringe.

Product intended for multiple use except the applicator which is for single use only. Multiple use of applicator may cause the risk of reinfection.

**CONTRAINDICATIONS**

Do not use the product for patients with hypersensitivity to the product's components.

**PRECAUTIONS**

It irritates the eyes.

Use rubber dams while working with the product. Use disposable gloves, protective glasses and clothing.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

**STORAGE**

Store in original packaging below 25°C. Keep out of reach of children. The expiry date is on the direct packaging.

Once container is open, the expiry date doesn't change if the container is securely closed after each use.

**HANDLING THE PRODUCT CONTAINER**

Empty containers should be disposed of responsibly or returned to the manufacturer.

**PACKAGING**

Syringe with 2 ml of preparation, disposable brush applicators.

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