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INSTRUCTION FOR USE

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY****COMPOSITION:** propane-butane, orange scent, alcohol.**INTENDED USE**

Spray for pulp vitality testing with orange scent.

INSTRUCTIONS FOR USE

Apply the product by spraying the target surface precisely and several times, from a distance of a few centimeters. Usually, a spraying time of 1-3 seconds is sufficient.

Attention: the direct contact with the liquid contents of the can through splashes or spray jets can cause frostbite on the surface of the skin and in the oral cavity.

PRECAUTIONS

Extremely flammable aerosol. Pressurised container: May burst if heated. Causes serious eye irritation. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Contains (R)-p-mentha-1,8-diene; d-limonene. May produce an allergic reaction. Refrigerated liquefied gas. Contact with the product can cause cold burns or frostbite. Even after use and until complete evaporation of the flammable components, there is still a danger of an explosive steam-air mixture forming.

Do not use if the person in treatment is allergic to any of the ingredients.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE AND TRANSPORT

Protect from frost, humidity and wetness.

Store in a cool, well-ventilated area.

In case of insufficient ventilation and/or through use, explosive/highly flammable mixtures may develop.

The relative humidity should be below 60%.

Protect from temperatures above 35°C/95°F.

Protect from open flames and direct sunlight.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.

Protect from crushing and squeezing.

It is not allowed to stack complete pallets unless we have explicitly allowed it.

Store in a secured place and keep out of reach of children.

Protect from highly fluctuating temperatures, like e. g. in a sea container.

Protect from high humidity and aggressive air, like e. g. sea air.

The ideal temperature for transport and storage is between 15°C/59°F and 25°C/77°F

Do not smoke nearby.

The goods have to be safely secured during transport to prevent slipping and falling over.

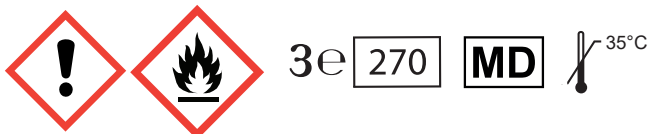
PACKAGING:

Spray 200 ml.

Expiry date and LOT number are placed on the main container.

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DEVICE REGISTERED UNDER THE NAME: DENTAL-KALTESPRAY ORANGE

Last update: 13.08.2021/3

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Friedrich Huber aeronova GmbH & Co.