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EN
<p>BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY</p>

COMPOSITION: aluminium chloride 25%, water Ph. Eur., menthol, excipients.

INTENDED USE
ALUSTAT is intended for use during dental treatment to staunch slight gingival bleedings.

Aluminium chloride in the product has the vasoconstrictive effect on tiny blood vessels, as a result of which the gingival swelling and slight bleedings decrease.

INSTRUCTIONS FOR USE
Rub gently the swollen or injured gingiva with a cotton wool tampon soaked in ALUSTAT and rinse with clean water. Repeat if necessary. Recommended contact time up to 5 minutes.

CONTRAINDICATIONS
Do not use for patients who are hypersensitive to the product components.

PRECAUTIONS
Irritating to the mucous membranes of the oral cavity, skin and eyes. In case of contact, rinse immediately with plenty of water and seek medical advice.

If swallowed, do not induce vomiting. Drink plenty of water. Seek medical advice. Use disposable gloves and protective glasses while working with the product. Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

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HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Bottle with 10 g of product and dropper.

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UK
<p>BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY</p>

COMPOSITION: 25% of aluminium chloride, water of pharmaceutical grade, menthol, excipients.

USE
The product ALUSTAT is intended for use during dental treatment to staunch slight gingival bleedings.

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CONTRAINDICATIONS
Do not use for patients who are hypersensitive to the components of the product.

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