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EN
<p><b>BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY</b></p>
<p><b>COMPOSITION:</b> disodium edetate (EDTA 17%), surfactants, water Ph. Eur., excipients.</p>
<p><b>INTENDED USE</b> Product ENDO-SOLUTION PREMIUM is intended for the mechanical root canal preparation. Disodium edetate (EDTA) contained in the product, rinsing calcium and magnesium ions, softens the surface layer of the dentin, facilitating its removal and restoration of the root canal. The product contains surfactants that improve the penetration of the preparation and accelerate the removal of the smear layer from the interior of the developed root canal, exposing root canal orifices. Adapter with a plug for taking the liquid preparations into the syringe 1. Unscrew the bottle cap. 2. Screw the adapter on the bottle. 3. In order to use the bottle, remove the blue plug, screw the end of the syringe into the adapter opening, tilt the bottle with the preparation upside down and take the liquid into the syringe, while simultaneously holding down the adapter. Next, put the bottle upright on the table top and gently unscrew the filled syringe from adapter. It is recommended to use luer-lock syringes with a rubber plunger. 4. After each use, close the adapter opening with a blue plug. 5. For transport, remove the adapter and screw the cap on the bottle.</p>
<p><b>INSTRUCTIONS FOR USE</b> Insert to open the root canal, insert 2-3 drops of ENDO-SOLUTION PREMIUM into the canal while developing the canal mechanically for approx. 30 seconds. Rinse with physiological saline. Repeat the process a few times. Rinsing with sodium hypochlorite (CHLORAXID) is recommended.</p>
<p><b>CONTRAINDICATIONS</b> Do not use the product for patients with hypersensitivity to the product's components. Do not leave the product in the canal or the tooth cavity between visits. This may result in excessive softening of dentin.</p>
<p><b>PRECAUTIONS</b> Extreme caution is advised when perforation is present. May cause severe skin burns and eye damage. In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting. Drink plenty of water. Seek medical advice. Use a rubber-dam when working with the product. Use protective gloves, glasses and protective clothing. <b>DISPENSER IS NOT INTENDED FOR STORING LIQUIDS!</b> Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.</p>
<p><b>STORAGE</b> Store in original packaging below 25°C. Keep out of reach of children. Expiry date is placed on the main container. Once the container is open, the expiry date doesn't change if it the container is securely closed after each use.</p>
<p><b>HANDLING THE PRODUCT CONTAINER</b> Empty containers should be disposed of responsibly or returned to the manufacturer.</p>
<p><b>PACKAGING</b> Bottle of 120 ml of preparation, adapter with a screw cap, dispenser.</p>
<p>Last update: 16.09.2021/7</p>

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